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### FFP2 Protection face mask with earloops

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#### Instructions for use

1. Before putting on the FFP2 mask, wash your hands with alcohol-based solution or with soap and water. 2. Remove the mask from the package, being careful not to touch the inside. 3. Make sure the adjustable nose bridge is at the top of the mask near your nose (not your chin). 4. Put the mask on your face. Pinch and press the nose bridge onto your nose and place the ear loops around your ears. 5. Make sure there are no gaps between your face and the mask. 6. Adjust the mask on your face by holding it from the back sides (not the front). 7. Avoid touching the mask while using it. 8. Replace when it is moist, damaged, visibly soiled or difficult to breathe through.

**For maximum fitting:** Cover the front of the mask with both hands, taking care not to impair the fit of the mask. Exhale strongly. If air leaks around the nose, re-adjust the nose clip to minimize leakage.

**Attention:** Before use, check the integrity of the package and the expiration date. Recommended maximum use: 6-8 hours. Discontinue use if you experience shortness of breathe, malaise or adverse reaction occur. Do not reuse disposable masks. Do not use if the package is opened or damaged.